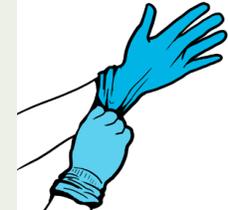


CONSERVE: N95 Respirator, Face Shield, and Goggles Personal Protective Equipment (PPE)

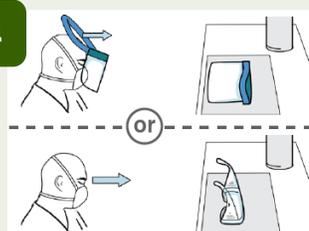
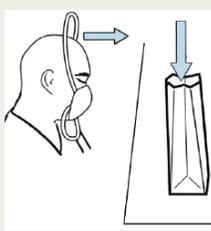
Prepare Area

After gown and glove removal:

<p>1</p>  <p>Sanitize hands.</p>	<p>2</p>  <p>Put on fresh gloves.</p>	<p>3</p>  <p>Place wipe on table.</p>
--	---	--

N95 Respirator

After following steps 1-3:

<p>4</p>  <p>Remove eyewear and place on wipe.</p>	<p>5</p>  <p>Sanitize gloves.</p>	<p>6</p>  <p>Remove procedure mask, lower strap and then upper. Store mask.</p>	<p>7</p>  <p>Sanitize gloves.</p>
---	---	---	---

Face Shield

After following steps 1-3:

<p>4</p>  <p>Wipe:</p> <ul style="list-style-type: none">• front and back of shield• elastic band• foam band	<p>5</p>  <p>Wipe table.</p>	<p>6</p> <p>Place shield on clean surface to dry.</p>	<p>7</p>  <p>Sanitize gloves.</p> <p>8</p>  <p>Remove gloves.</p>	<p>9</p> <p>Wash hands with soap and water.</p> 
---	--	---	--	--

Goggles

After following steps 1-3:

<p>4</p>  <p>Wipe:</p> <ul style="list-style-type: none">• front and back of lens• both ear pieces	<p>5</p>  <p>Wipe table.</p>	<p>6</p> <p>Place goggles on clean surface to dry.</p>	<p>7</p>  <p>Sanitize gloves.</p> <p>8</p>  <p>Remove gloves.</p>	<p>9</p> <p>Wash hands with soap and water.</p> 
--	--	--	--	--

* It is important to always confirm your sanitizer is compatible with your PPE. If you are unsure, test first or consult your PPE specialist. Incompatible sanitizer can compromise the integrity of your PPE.